

Theramag VF 30P

THE APPARATUS FOR STIMULATING THERAPY



Introduction

Dear valued customer,

Thank you for the confidence you have placed in us by purchasing the Theramag VF 30P device. By making this decision you have joined thousands of satisfied customers who spread the idea of improving one's health in the comfort of one's home.

This guide is put together in such a way so as to provide you with answers to all important questions associated with the use of Theramag VF 30P. If you require any assistance, please do not hesitate to contact us at any time via the phone number or email address listed at the back of this manual.

Important note

To become eligible for the extended warranty for your device UltiCare LT-99, it is necessary to register the device within 30 (thirty) days of purchase. Registration will provide you with an extended warranty with the duration of 12 + 24 months.

Please go to the website: <http://www.orin-us.com> and click on "Register my device"

Thank you very much.

Yours sincerely,

ORIN US LLC.

orin CONTACT US DIRECTLY SIGN IN REGISTER 0.12 / 0 €

CHOOSE A DISEASE FROM THE LIST ...

HEALTH PROBLEMS REFERENCES PRODUCTS EDUCATION ABOUT ORIN REGISTER MY DEVICE

Register my device

Please ensure you register this product within 15 days from the day of purchase or ORIN US will have no responsibility of any limited damage from usage or having consequences from improper usage.

S/N: 00023 Mobile number:

Date of purchase: April 29th, 2015 Email:

First name:

Last name:

Address line 1:

Address line 2 (optional):

City:

State / Province / Region:

Postal code:

Register my device

Contents

Introduction, Important note	2
Overview	4
Disclaimer	4
Safety precautions	5
What is stimulating therapy	5
The benefits of the product	6
NTS (Never the same) modulation protected by European patent	7
Contraindications	7
Product description	9
Technical data	11
Operating modes (programs)	11
Installation of the device and its operating	12
Notes for the dual program mode	14
User set-up of program parameters	14
Device Setup mode	15
Error signals and messages	16
Product properties and handling	16
Warnings	16
Maintenance and repair	17
Notes	18
Program list	19
Limited Warranty and Policy for Return of Products	26
Returns policy	27

Overview

The Theramag VF 30P and associated external applicators are a range of devices which stimulate human body by the application to the affected part of a pulsed magnetic field (PMF).

A patented mechanism ensures that the field varies in a random way which has been shown to improve the effectiveness of this kind of stimulating therapy compared to devices delivering a repeating of frequency variation.

This brochure aims to explain how PMFT works and precisely how to use the Theramag for maximum benefit.

The device is easy to use, as is demonstrated pictorially in this brochure.

Disclaimer

Theramag VX 30P device is registered and regulated by the Food and Drug Administration (FDA), however they have not been evaluated by the FDA to cure or prevent any disease. Theramag device is intended for the beneficial effects as set forth in the instruction manual and product literature. This one device help the body to stay healthy and promote general wellness.

You are advised to consult with your physician and healthcare provider before using Theramag device if you have any concerns regarding your health. As a Theramag product user, you understand that you are assuming full responsibility for the safe and proper use of the device, and you will only use the Theramag device in accordance with usage information set forth in the instruction manual and product literature, as well as on the www.orin-us.com website.

Further, this will constitute an implied indemnity that you, as the buyer of the Theramag device agree to indemnify and hold harmless the supplier and manufacturer of the Theramag device from any consumer claims against this one product or its ultimate use.



Safety precautions

! Prior to using this device, read a follow all of operating instructions in User Manual.

! Use this device only for its intended purpose as described in User Manual.

! Use this device only with the power supply provided by the manufacturer.

! Do not use this device if it has damaged cord or plug, or if it is not working properly.

! No parts of this device should be exposed to direct sunlight, intense heat or humidity.

! Do not use this device in a wet environment or in a bath. Do not submerge the device in water.

! Children should only use this device under close adult supervision.

! Only qualified service personnel should service this device. In no case should the front panel or the applicator be opened.

! In case of device failure, please contact your seller first or orin@orin-us.com. This device contains electronic components that cannot be serviced or repaired by the consumer.



What is stimulating therapy

Stimulating therapy is a non invasive holistic approach to wellness that fine tunes the body at the cellural level. Ultimately helping the body to heal and restore natural balance.

In the past, people knew the static magnetic field stimulating effect and used the magnetism of certain rocks (e.g. loadstone) and later of artificial, permanent magnets. Electronic devices that produce a low-frequency pulsed magnetic field with a strong stimulating effect have been designed more recently. This low-frequency magnetic field is completely harmless for humans as well as for animals; it shows a positive effect on all living organisms.

The pulsed magnetic field favourably influences the biochemical and biophysical reactions in the cells and between the cells in a living body, it improves the penetration

of a cell membrane that may be out of balance due to illness. As a result of this process, the cells are better supplied with oxygen and other helpful materials and the metabolic process improves. The effect on the cells is noted by the central nervous system, which starts self-healing processes in the body through the activation of the body's immune system.

From the above-mentioned facts, it is obvious that stimulating therapy is suitable as an additional method to regular treatments and is more and more frequently gaining a reputation in rehabilitation.

Theramag VF-30P generates a specific pulsed magnetic field that directly supports the regular treatment in patients and affects their physical and psychological well-being favourably.

Theramag beneficial effects



- Reduce muscle tension - myorelaxing effect
- Improve circulation - better blood supply to the tissues
- Has anti-inflammatory effects - reduce inflammation
- Suppresses swelling
- Reinforces and stabilizes bone tissue
- Improve cell function
- Improve the absorption of nutrients
- Stimulate the immune system
- Reduce stress
- Balance acupuncture meridians
- Improve sleep
- Improves cell and tissue metabolism
- Help the body to detoxify
- As a result of all these effects, it reduces the use of drugs, speeds up the recovery process and reduces or eliminates symptoms in patients with chronic diseases

NTS (Never the same) modulation protected by European patent

In order to ensure maximum effectiveness (speed and action) of the stimulating therapy, the ORIN stimulating devices have been designed so that the generated magnetic pulse frequencies are never repeated in the same pattern. This patented technology is unique and is available only in Orin devices. [European Patent number EP1500410](#).

The European technology

Stimulating therapy based on Pulsed Magnetic Field technology has been used in Europe since 1980.

Approved as active non - invasive medical device Class IIa in Europe

A natural patch to wellness

Non invasive approach designed to result in biological coherency and sufficient synchronization of ongoing metabolic processes intended to restore natural balance and bring the body back to a natural state of wellness

Flexible and easy to use

Thanks to its easy portability, the unit is ready to help its owner on trips and journeys. Simple to operate in both the office and home

Contraindications

Theramag VX 30P should be avoided during pregnancy, although no clinical studies have given evidence of any unwanted effects of this treatment. This is just a precautionary measure. In no case should the product be used by patients with any electronic implants (such as pacemakers or insulin pumps). The effects might be fatal.

Bleeding of any kind is also a condition in which stimulating therapy is not recommended because magnetic field slightly suppresses blood





coagulation and promotes blood supply to the tissues, whereby bleeding is supported. Stimulating therapy is contraindicated in patients with bleeding into the digestive tract.

The use of the device should be stopped two days before, during, and two days after menstruation, although a pulsed magnetic field may help reduce menstrual pain if applied outside this time period.

Stimulating therapy is not recommended for patients with acute viral diseases, heavy fungous diseases, neurological seizures or severe atherosclerosis.

Do not use stimulating therapy in patients with adrenal hyperfunction, thyroid hyperfunction, active tuberculosis, venous thrombosis or embolism, hypothalamus or hypophysis disorders. All the above information is given as a precautionary measure.

Also as a precaution, avoid using the product to treat persons with existing or past malignant tumours, even if treated and supposedly eliminated.

Based on current knowledge, stimulating therapy does not have any side effects if the recommendations given here are followed.

In extremely sensitive individuals, use of the system can induce an appreciable temporary blood pressure drop, this response, however, usually vanishes after the first five applications.

How should the device be used and for how long?

Theramag is physiotherapeutic device intended for medical professional usage and within health service communities i.e. home health care, in physiotherapy, rehab facilities and med-spa.

There is no specific training needed for operating of the device. Theramag can be operated by person above 18 years, who is able to understand to this manual. Slightly impaired sight, hearing and mind of user do not matter, but ability of desired operation of device is required. The use of the system several times a day is absolutely safe, however daily exposure should not exceed one hour.

Product description



1 – mains socket
 2 – mains switch
 3 – USB connector

4 – lift-up panel with display and indicators
 5 – group of control buttons
 6 and 7 – applicator outputs



A4C applicator



A1C applicator



B1C applicator



APE applicator



Technical data

- Type designation: VF 30P
- Power supply: 100 - 240V, 50/60 Hz
- Maximum power consumption of the generator: 60 VA
- Two outlets for applicators
- Magnetic induction: up to 15 mT, depending on the type of applicator used
- Repeating frequency of pulses: 1 to 99 Hz, according to the program chosen
- Operating temperature range: +5 to +35 °C
- Operating relative humidity: 15 to 93 %, non-condensing
- Operating atmospheric pressure: 700 to 1060 hPa
- Storage and transport temperature range: -40 to +70°C
- Storage and transport humidity: 10 to 100 %
- Storage and transport atmospheric pressure: 700 to 1060 hPa
- Weight of the generator - control unit: 1.15 kg



Operating modes of the device:

There is possible to operate device in four modes.

Single frequency: Mode, where pulsed magnetic field of fixed repeating frequency of pulses is generated. This frequency can be set with 1 Hz step in range from 1 Hz to 99 Hz.

Wobbling frequency: In this mode pulsed magnetic field with repeating frequency wobbled in selected range is generated. Low and high frequency limit can be set in range from 1 Hz to 99 Hz, with 1 Hz step.

Random frequency: In this mode pulsed magnetic field with randomly varying repeating frequency in selected range is generated. Low and high frequency limit can be set in range from 1 Hz to 99 Hz, with 1 Hz step.

Pulse burst: Mode, where groups of pulses of selected frequency are generated. There is possible to set 2 to 20 pulses in group, with repeating frequency from 1 Hz to 99 Hz.

Stimulating programs may operate in any of above mentioned operating modes.

There is set of 4 universal stimulating programs (marked P1 to P4) preset by the manufacturer as default. Their parameters are described in the table below.

User can expand number of programs up to 30 in the DEVICE SETUP mode of the device (see instruction below) and redefine parameters of each program according to recommendation mentioned in the Program appendix or according to the physiotherapists recommendation.

Preset universal therapeutic programs


Program	Operating mode	Application time	Frequency	Intensity
P1	wobling	20 min	4 - 40 Hz	50%
P2	random	20 min	2 - 72 Hz	50%
P3	single freq.	20 min	12 Hz	50%
P4	pulse burst	20 min	5 imp. / 24 Hz	50%

Components of the VF-30P unit

1. VF-30P control unit (generator)
2. Applicators (applied parts) A1C, A4C, APE and B1C are supplied according to the customer's requirements
3. Magnetic field indicator
4. Power supply cord 2.5m /2x0.75mm²

Installation of the device and its operating

Place control unit (1) on the suitable table within reach of mains (up to 2 meters). Connect supplied power cord (4) into the socket on the back side of the device. Set mains switch to the off state (position 0). Then insert power cord into the mains socket and turn on device using mains switch (position 1). Also you can lift up the display panel. If there is

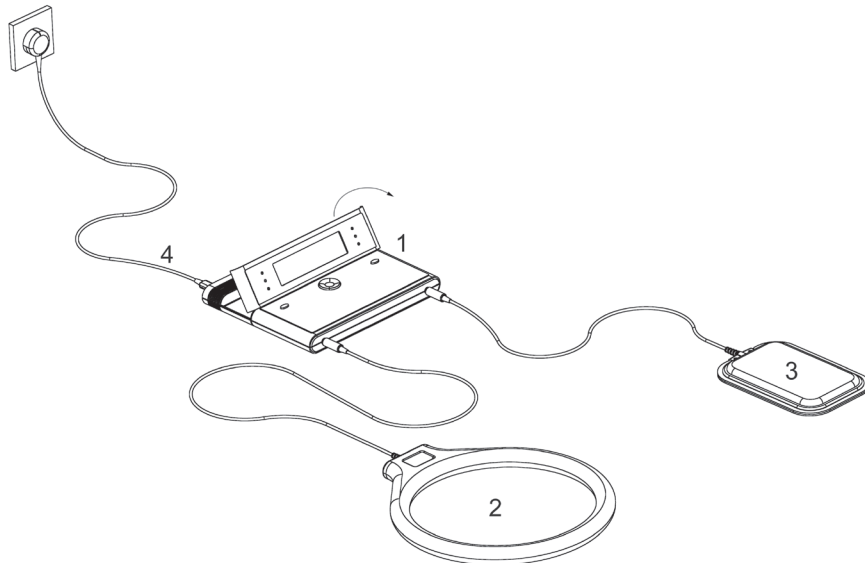

mains voltage present, the POWER indicator will light up, welcome text become present on display and short test indicated by blink of all indicators is performed. By pressing any key device pass into initial state, where the last used program is selected and its parameters are displayed. At first usage of the device, it is program P1.

Outputs for applicators, marked OUT.1 and OUT.2 on the device panel are equivalent. Connect one or two applicators (2, 3). If the connection is good, indicators near output sockets light-up green. Device is ready to run now.

Run device by short press of OK button. Using the same button device can be paused. Running of the device is signaled by yellow indicator RUN, pause is signaled by green indicator PAUSE.

In the initial state or in the pause state there is possible to select another program using ▲ or ▼ button and start it by above mentioned way.

You can also use another program on each output (dual program mode). If you wish this, press ◀ or ▶ button. Left or right PRG.N text becomes highlighted. Now, using ▲ or ▼ button you can select program for left or right output. By pressing OK or CTRL1 or CTRL2 you can run programs on both or on one output.

If you wish to set the same program on both output, press ◀ or ▶ button until no left or right PRG.N is highlighted. Now press ▲ or ▼ button, and programs on both sides

become the same, and changing simultaneously.

Notes for the dual program mode

- Time is the same for both outputs. If you start both outputs simultaneously, using OK button, the longer time from the both times will be used.
- If you start only one output (using CTRL 1 or 2), and later you start the second output, it will use the same time as the program on the side, which was started as first, have defined.
- While device is running, you can use ◀or▶ button to select left or right output, and see the parameters of the corresponding program.
- Sound mode of the device is set regarding to the program running on the selected output. It means so if you have selected (highlighted) program for left output, you will hear its pulses, if FULL sound is enabled. Same for the right output.
- You can overcome actual sound mode using ▲ or ▼ button while running. You can set lower level of sound (only 5min. beep or no sound) regardless of setting of actual running program. This setting is temporary, only for the actual run of the program and is signaled by graphic symbol, which is visible only when at least one output is running.
- You can temporary change magnetic field intensity for each output. In both (stop and running) modes, hold CTRL1 or CTRL2 button, until INTENSITY item is highlighted. Then set new value and press OK. This setting is temporary, only for the actual run of the program.

User set-up of program parameters

User can change parameters of currently selected program, according to which device generates pulsed magnetic field. There is possible to change mode and the corresponding range of repeating frequency of pulses, time of application, magnetic field intensity and enable/disable sound during application.

After selecting of program that will be modified press and hold OK button for entering into the setup mode. It is signaled by the text SETUP beside to program number. Also, there is highlighted parameter, which can be modified using ◀or▶ button and confirmed by short press of OK button. Use ▲ or ▼ button for selection of another

parameter to be modified.

Exit from setup mode can be done by long press of the OK button. Text SETUP disappears and all performed changes in program parameters are stored now, and every next run of program will be with new parameters.

By using of CONTROL 1 and CONTROL 2 buttons, there is possible to work individually with OUT.1 or OUT.2 output. By short press we can run or pause program on corresponding output. By long holding of CONTROL button there is possible to temporary modify intensity for corresponding output.

Return to the default settings of parameters, as they are defined by the manufacturer, can be performed in the DEVICE SETUP menu.

Device Setup mode

This mode can be entered by holding the OK button when switching-on the device. After doing this, device comes into set-up mode, signaled by DEVICE SETUP text on display. Also, there is actual version of the firmware displayed. There are five items available in the device setup menu. Three of them (in the left column) are accessible for user:

- MAX. PROGRAM COUNT: this item defines number of programs offered to the user. Minimum number of programs is 4 (this is default setting) and can be expanded up to 30 programs as maximum.

Note: Decreasing this number does not lead to erase of definition of programs, which are not accessible after decrease of program count. Programs stay stored in memory of the device and after increase of this parameter are enabled and accessible again.

- DISPLAY BRIGHTNESS: this item defines brightness of the display and can be set in range 30 to 100%.
- RESET TO DEFAULTS: setting this item to YES value, followed by pressing of the OK button, leads to reset of the programs definitions to the state as they were defined by the manufacturer (programs P1 to P4, see table above).

Two following items (SERVICE MENU and TOTAL RUN TIME) are only for manufacturer's purpose and access into these sections is protected by the password. Device setup mode can be terminated by selecting EXIT SETUP item and pressing OK button.

Error signals and messages

Errors of output modules and errors of applicator circuits are signaled by the appropriate text message or symbol on the bottom line of the display. Significant errors are signaled also acoustically.

Product properties and handling

The applicators A1C, A4C and ASE act on the stimulated body parts through clothing, do not apply them to bare skin from hygienic reasons. If application to the clothing is impossible for any reason, put a piece of cloth on the site to be stimulated.

The applicators B1C (torus) and APE (pen) are made of ABS copolymer, which meets the requirements for contact with the patient's skin.

No hazardous substances are contained in the product or used by the product during operation.

When the service life has expired or the product is no longer usable, it should be handled as electric waste, i.e. either returned to the manufacturer/dealer for disposal free of charge or collected as special waste in compliance with applicable legislation.

Applicators

-

so called applied parts of VF 30P device

There is an electromagnetic system (consisting of one or more electromagnetic coils) placed in the body of the applicator. Current pulses fed to the applicator from control unit generates pulsed magnetic field around the applicator. Parameters of this magnetic field are determined by the construction of the applicator and selected program. Applicators A1C, A4C, B1C and APE are VF type, thus supplied with coaxial cable of 7,874 ft length, terminated by the 0,248 inch jack connector.

WARNINGS

- Protect the device against moisture (defined by symbol on the instrument label). Use device only in indoor dry places.

- Use only applicators supplied by ORIN US for this product, otherwise there is not electromagnetic compatibility (EMC) guaranteed.
- In case of power supply cord replacement, use cord with the same parameters, including length. Otherwise there is not safety and electromagnetic compatibility (EMC) guaranteed.
- Modifications of the device are forbidden.
- Wireless communication devices (such as mobile phones, cordless phones, handie-talkies, home wireless network devices) must not be placed closer than 2 ft. from device (including applied parts) to prevent interference.
- Do not connect an external applicator to the unit or disconnect it from the unit while the unit is running. Connect/disconnect an external applicator before starting the unit or while operation is suspended (PAUSE). Otherwise, the output circuits of the system could be damaged.
- Put the system out of operation only by unplugging the power supply cord from the wall socket.
- The product meets the requirements for electromagnetic compatibility. Nevertheless, remember that the system forms a magnetic field of one polarity and very low frequency, which can (similarly to a permanent magnet) affect some instruments and facilities located in the immediate vicinity (such as computer monitors and TV screens, classical watches, bank cards with magnetic strips etc.).

Maintenance and repairs

The system does not require any special maintenance (such as aligning, calibration, battery replacement etc.). You can make a tentative performance test by using the magnetic indicator supplied with the product.

Clean the plastic applicators B1C and APE with a cloth wet in water and detergent. Use vacuum cleaning or brushing to clean the A1C, A4C applicators. The unit components must not be submerged in water for cleaning.

Report any product failure to your dealer's servicing department at first. If this is not possible from any reason, then contact manufacturer.



Notes

Multiple daily exposure should be spread out the whole day.

Application regiment values are for your orientation. In some very rare cases an impairment may occur especially in magnetostimulation of inflammatory conditions. This is no reason to stop the stimulating therapy but the application should be changed typically by frequency reduction to 50% of the initial value. The total dosis can be reduced by application time reduction or by both values. When an improvement does the reduced values may increase again.

If you use our device regularly there are data showing that pain attacks can be eliminated for considerable long time.

In sport medicine the stimulating therapy has amplified effects of comonly used rehabilitation and regeneration procedures.

The stimulating therapy as any other kind of physiotherapy represents a weak stimulus. That's why it must be applied for a long time and regularly. E.g. positive effects in painful and inflammatory conditions may come after three days of application but there are rare cases where the first signs of improvement were reached after two weeks of daily application. In cases of slow fractures healing the time required for an effect is still longer.

When the problems reduce or disappear, this fact does not mean that the therapy should be stopped suddenly. Much better is to go on with the same regimen for another week and then reduce the number of applications stepwisely, e.g. Monday, Tuesday, Friday. If minimum of the problems returns then start with regular stimulating therapy again for at least one week.

Confidentiality Statement: *The content of this manual is considered privileged and to be kept strictly confidential. If the reader is not the intended recipient, or one of its registered partners, be advised that any perusal, use, dissemination, distribution and/or copying or sharing it's content is strictly prohibited without written consent from Orin. If you have received this manual/document in error please destroy the manual and notify Orin-USA immediately.*

PROGRAM LIST

STIMULATING OF BACK

Lower back pain	P3/28 Hz/W, 2 x 20 min. per day, minimally 10 days	See the picture 1
Chronic back pain	P1, 3 x 20 min. per day	See the picture 2 or 3
Catatonia of the back	P1, 3 x 20 min. per day	See the picture 1
Ankylosing spondylitis	The first three exposures up to P3/10 Hz, then P3/15 Hz/W, 1 x 20 min., minimally 15 days	See the picture 1



Theramag
PMT BIOSTIMULATOR Model VF 30P

STIMULATING OF JOINTS

Joints aches and injuries	P1 or P3/12 Hz/W, 2x20 min. per day	See the picture 1
A help with weather - related rheumatic pain	P1 or P3/15 Hz/W, 3x20 min. per day	See the picture 1
Reduce mobility of small hand joints	P1 or P3/18 Hz/W, 2 x 20 min. per day	See the picture 2
Stimulating of swelling of large joints	P1 or P3/22 Hz/W, 1 x 20 min. per day	See the picture 3
Osteoarthritis, knee & hip joints, other joints	P4 or P3/19-25 Hz, 2 x 20 min per day	See the pictures 1 - 3
Tennis elbow or frozen shoulder	P3/10 - 17 Hz, 2 x 20min. minimally 10 - 15 days, the first application on the elbow / shoulder, next application along the spine	See the pictures 4, 5, 6



STIMULATING OF BONES

Fresh fractures and other bone related illnesses	P4 or P3/15-25 Hz/W, 4 x 20 min. per day	See the picture 1
Calcification improvement	P3/70-72 Hz/W, 2 x 20 min. per day	See the pictures 1 - 3
Sport and occupational injuries where bone structures and joints are involved	P3/15-25 Hz/W 4 x 20 min. per day	See the picture 2



Theramag
PMT BIOSTIMULATOR Model VF 30P

STIMULATING OF IMMUNE SYSTEM

Helps increase the body's immune strength and natural resistance to infections	P2, 2 x 20 min. per day, 14 days	See the picture 1
As prevention during epidemic periods	P2, 2 x 20 min. per day, 14 days	See the picture 1
Metabolic improvement	P1 or P4, 2 x 20 min. per day, 14 days then 14 days off	See the picture 2
Effective for all ages	P2, 2 x 20 min. per day, 14 days	See the picture 1



HEELS

Stimulating of heels	P2 or P3/28-30 Hz/W, 2 x 20 min per day	See the picture below
----------------------	---	-----------------------



STIMULATING AFTER SPORT

Can be used after workouts and daily physical activities	P1, immediately after training or competition	See the pictures below
Supports recovery and regeneration processes after physical exhaustion	P1, immediately after training or competition	See the pictures below
Aids in the recovery of muscle tissue after intense training or competition	P1, immediately after training or competition	See the pictures below



MUSCLES

Aids in repair of damaged muscle tissue	P3 22 Hz/W, or P4, 1 x 20 min per day	See the pictures below
Aids for painful muscles	P2 or P4, 2 x 20 min per day	See the picture below
Fibromyalgia	P2 or P4, 2 x 20 min.	See the picture



NERVES

Stimulating of pain syndromes (carpal tunnel)	P1 or P3/10 - 15 Hz/W, 2 x 20 min per day	See the picture below
---	---	-----------------------



URINARY

Stimulating of prostate	P1 or P3/22-27 Hz/W, 1 x 20 min per day	See the picture below
Enuresis nocturna	P3/25 Hz/W, 1 x 20 min per day	See the picture



CIRCULATION

Circulatory disturbances	P4 or P3/15-20 Hz/W, 2 x 20 min per day	See the pictures 1 - 3
--------------------------	---	------------------------



YANG

Poor libido	P4 or P3/15 - 18 Hz/W, 2 x 20 min per day	See the pictures 1 - 2
-------------	---	------------------------



YIN

Discomfort during menstruation	P1, 2x20 min per day	See the picture below
Regulation of menstrual cycles	P1, 2x20 min per day	See the picture below
Pains following ruptures of ovarian cysts	P1 or P3/15 Hz/W, 2x20 min per day	See the picture below



WARNING: Theramag can not be used 3 days ahead of the menstruation and 3 days after it

Limited Warranty and Policy for Return of Products

We at Orin US LLC are proud that you have selected one of our unique products to help you on your way toward more healthful living. We truly appreciate your business. The BioTorus LT-100, and all other devices manufactured or marketed by ORIN US LLC and its affiliates, are warranted against material and manufacturing defects only. This is a limited warranty provided to the original purchaser with verified proof of purchase. This device is a complex electronic device and is sensitive to our unique calibration designed for optimal use and effectiveness. We have established specific rules and specifications that we use to evaluate and repair any claimed defect considered by this warranty. Only authorized technicians from ORIN US LLC and its affiliates are permitted to perform any repair on the device or its parts. Any unauthorized evaluation or repair of the device

immediately voids this warranty.

ORIN US LLC, and its affiliates, agents, principles and employees, denounce any and all liability for equipment effectiveness, reliability and safety in the event that any modification, evaluation or repair is carried out by an unauthorized person or if the device is used other than as specified in the user manual provided with the device.

The BioTorus LT- 100 carries a 12 months limited warranty and is limited to devices returned for claimed material defect caused by the manufacturer. If you register at www.orin-us.com per instructions on page „Register my device“ of this manual your warranty will be extended by additional 24 months, which means the total warranty for the BioTorus LT-100 device will be 36 months.

In order for the limited warranty to apply to any purchased product, the consumer must register the product using the registration materials provided with the product or as described in the user manual. The limited warranty shall begin to accrue beginning on the date the product is received by the consumer. Product registration must be received by ORIN US LLC within 30 days after receipt by the consumer. Registration of the product shall be considered as understanding and acceptance of the warranty terms and conditions by the consumer.

Return Policy

At ORIN US LLC we do our very best to provide products that make the lives of our customers happier and healthier. We accept that sometimes products must be returned to us for a variety of legitimate reasons. The guidelines below will insure the fastest possible turnaround should you need to return an unused product.

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
The consumer may return any undamaged product within 30 days of the Delivery Date and receive a full refund of any payments made toward the Purchase Price in full.

ORIN US LLC will process the refund within 15 days after receipt and evaluation of the returned product as long as it meets the criteria set forth above.

Of course, any disposable product or a product that has been used or expired cannot be returned except as stated in the terms of the limited warranty above.

In any case, shipping charges associated with the purchase will not be refunded and the customer will be responsible for return shipping charges. No refund will be issued for damaged or marred goods returned, including but not limited to products that have been marred by inundation with the smell of tobacco or other persistent odor reasonably rendering the product unsaleable.

Theramag VF 30P



ORIN US, LLC.

1-727-512-5074
orin@orin-us.com
www.orin-us.com



Manufacturer:
Made in
European Union



DO NOT USE
IF PACKAGE
IS DAMAGED



40°C
UPPER
LIMIT OF
TEMPERATURE



KEEP DRY

